
## Homedics quick-start guide

**XL** UPPER  
ARM CUFF

Works exclusively with the following Homedics branded blood pressure monitors:

- 500 Series / BPA-900BT-WT
- 700 Series / BPA-100TRBT
- BPA-960BT-WT / 1565256
- BPA-960BT
- BPA-945


BPA-CUFFXLA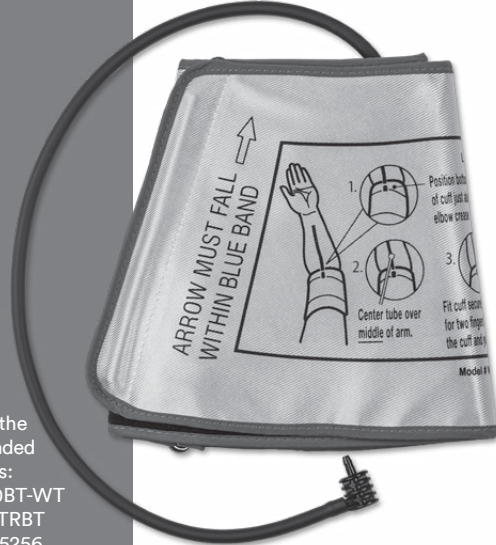


### Applying the cuff

**NOTE:** Wearing the arm cuff properly is critical for accurate measurements. If the circumference of the upper arm differs largely from the circumference of the forearm, an excessive gap may occur between the arm and the arm cuff, which could result in an inaccurate measurement. Please consult your health care professional if you have questions on how to properly wear the arm cuff.

1. If the cuff is not assembled, pass the end of the cuff furthest away from the tubing through the metal D-ring in order to form a loop. The smooth side without the felt material should be on the inside of the cuff loop.

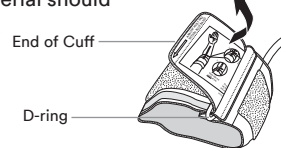
2. Plug the cuff tube into the left side of the unit.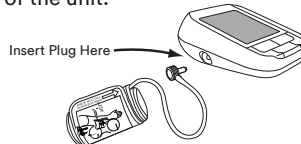


3. Open the arm cuff so that your arm may be placed through it.
4. Remove tight-fitting clothing from your upper left arm so that the cuff can be wrapped around your exposed arm.  
**NOTE:** It is recommended to wrap the cuff on a bare arm or over thin clothing to ensure accuracy. Thick clothing, or a rolled up sleeve, will cause inaccurate blood pressure measurements.

5. Position the cuff just above the elbow crease.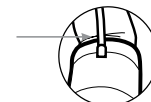


6. Center tube over middle of arm.

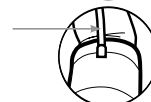7. Pull the end of the cuff so that it tightens evenly around your arm, making sure that the arrow falls within the blue colored line. Press the hook and loop material together to secure, ensuring the cuff does not slip during measurement. Allow room for 2 fingers to fit between the cuff and your arm.

8. Lay your arm on a table (palm up) so the cuff is at the same height as your heart. Make sure the tube is not kinked.

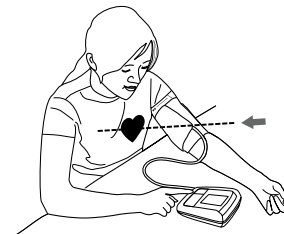9. Refer to the Instruction Manual included with your Homedics blood pressure monitor for further instructions on how to operate.

Questions or comments? Contact Homedics Consumer Relations at:

Email:  
cservice@Homedics.com

Phone:  
1-800-466-3342

Business Hours:  
8:30am-7pm EST  
Monday-Friday

#### LIMITED 2-YEAR WARRANTY

Homedics sells its products with the intent that they are free of defects in manufacture and workmanship for a period of 2 years from the date of original purchase, except as noted below. Homedics warrants that its products will be free of defects in material and workmanship under normal use and service. This blood pressure arm cuff meets the simulated measurement cycles test requirement per EN1060-3, part 8.10. This warranty extends only to consumers and does not extend to Retailers.

To obtain warranty service on your Homedics product, contact a Consumer Relations representative. Please make sure to have the model number of the product available. Homedics does not authorize anyone, including, but not limited to, Retailers, the subsequent consumer purchaser of the product from a Retailer or remote purchasers, to obligate Homedics in any way beyond the terms set forth herein. This warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; improper installation; unauthorized repairs or modifications; improper use of electrical/power supply; loss of power; dropped product; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Homedics.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adoption to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. THERE SHALL BE NO OTHER WARRANTIES EXPRESS OR IMPLIED INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS OR ANY OTHER OBLIGATION ON THE PART OF THE COMPANY WITH RESPECT TO PRODUCTS COVERED BY THIS WARRANTY. HOMEDICS SHALL HAVE NO LIABILITY FOR ANY INCIDENTAL, CONSEQUENTIAL, OR SPECIAL DAMAGES. IN NO EVENT SHALL THIS WARRANTY REQUIRE MORE THAN THE REPAIR OR REPLACEMENT OF ANY PART OR PARTS WHICH ARE FOUND TO BE DEFECTIVE WITHIN THE EFFECTIVE PERIOD OF THE WARRANTY. NO REFUNDS WILL BE GIVEN. IF REPLACEMENT PARTS FOR DEFECTIVE MATERIALS ARE NOT AVAILABLE, HOMEDICS RESERVES THE RIGHT TO MAKE PRODUCT SUBSTITUTIONS IN LIEU OF REPAIR OR REPLACEMENT.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on internet auction sites and/or sales of such products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate as to any products or parts thereof which are repaired, replaced, altered, or modified, without the prior express and written consent of Homedics.

This warranty provides you with specific legal rights. You may have additional rights which may vary from state to state. Because of individual state regulations, some of the above limitations and exclusions may not apply to you.

Distributed by  
**Homedics**

Homedics, LLC  
3000 N Pontiac Trail  
Commerce Township, MI  
48390

Made in China

©2022 Homedics, LLC. All rights reserved.  
Homedics is a registered trademark of Homedics, LLC.

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©2022 Homedics, LLC. Tous les droits sont réservés.
Homedics est une marque déposée de Homedics, LLC.